
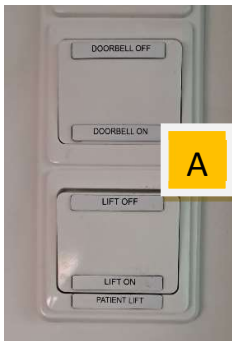


## Patient Lift – Type 1 – Standard Operating Procedure

1. Ensure the patient lift is turned on (A).
2. Open the safety lock (B).
3. Direct patient and/or gurney onto the lift and ensure the wheelchair/gurney brakes are on
4. Close gate locks (B), please note that the lift will not move unless the safety lock is fully engaged.
5. Push the 'up' button the remote controller (C) or lift control (D) and hold until the lift stops.
6. Open the scan room door by pushing the button on the inside of the trailer  
Note: The door will not open unless the lift is in its highest position.

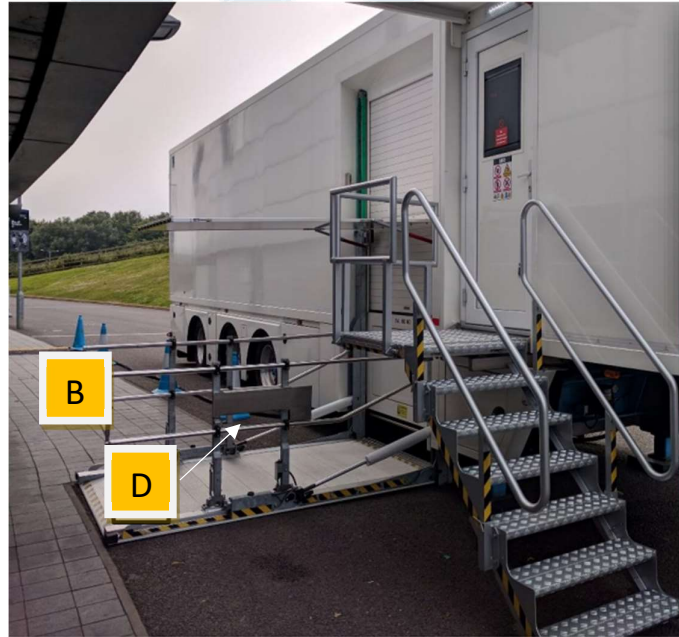

This Standard Operating Procedure (SOP) is provided solely for informational purposes and easy guidance. It is not intended to replace or supersede any instructions, recommendations, or safety warning provided by the manufacturer of the equipment referenced herein, Users of this SOP are reminded to consult and adhere to the manufacturer's manual and all associated safety guidelines and warnings before operating the equipment. The information provided in this SOP is no exhaustive, and users are responsible for ensuring compliance with all relevant safety regulations and standards. The organisation assumes no liability for any loss, damage, injury, or other consequence resulting from the use or misuse of this SOP, or any equipment referenced therein.